ARTHROSAMID[®]

Patient Information Leaflet

Arthrosamid®

en	4
cs	8
da	12
de	16
el	20
es	24
fi	28
fr	32
is	36
it	40
nl	44
no	48
pl	52
pt	56
sv	60
tr	64

For treatment of knee osteoarthritis.

Patient information leaflet

This brochure provides information on Arthrosamid®, an injectable non-absorbable hydrogel for treatment of knee osteoarthritis.

Please read the entire brochure and discuss it with your physician.

Contents

Glossary	4
What is Arthrosamid®?	4
What is osteoarthritis?	5
How is knee osteoarthritis normally treated?	5
How is Arthrosamid® administered?	5
How does Arthrosamid® work?	6
When should Arthrosamid® not be used?	6
What are the benefits?	6
What are the risks/ side effects?	6
Precautions	6
What can I expect after the treatment?	7

Glossary

Arthrosamid® is an injectable, non-absorbable and bio-compatible hydrogel.

Precaution

A statement in the product information that alerts the physician to take measures to avoid a problem.

Viscosupplementation

A procedure in which a gel-like fluid (e.g., hyaluronic acid) is injected into the knee joint. The fluid acts as a lubricant to enable bones to move smoothly over each other and as a shock absorber for joint loads.

Bakers' cyst

A fluid-filled growth behind the knee that causes a bulge and a feeling of tightness. It sometimes causes pain which worsens with activity or when fully straightening or bending the knee.

Synovial tissue

A layer of connective tissue that covers the inner joint capsule of the knee.

What is Arthrosamid®?

Arthrosamid® is a hydrogel consisting of 97.5 % sterile water and 2.5 % polyacrylamide. The hydrogel is supplied in a sterile pre-filled 1 mL syringe. Recommended dosage is 6 mL.

Arthrosamid® is an injectable, non-absorbable and biocompatible hydrogel intended to be used for symptomatic treatment of adult patients with knee osteoarthritis. The hydrogel will remain in the knee for the entire lifetime as an integral part of the synovial tissue of the inner joint capsule of the knee.

What is osteoarthritis?

Osteoarthritis is a long-term condition which affects the joints and worsens over time, causing pain and disability. It is most common in the knee joint where it wears away the shock-absorbing cartilage, causing bones to rub together and the joint to become stiff, swollen, and painful.

How is knee osteoarthritis normally treated?

The cause of osteoarthritis is unknown. There is no permanent cure, and treatments have focused on managing the symptoms. They include:

- Physical therapy and exercise.
- Weight loss.
- Pain relief medication.
- Knee injections with viscosupplementation and/or corticosteroids.
- Surgery such as knee replacement.

Arthrosamid® is intended for use in patients that still have symptoms after conservative treatment with physical therapy, exercise, weight loss and pain relief medication.

Your physician will advise you whether treatment with Arthrosamid® will be suitable for you.

How is Arthrosamid® administered?

Arthrosamid® is given as a single injection into the knee's intra-articular cavity by a qualified specialist – either an orthopedic surgeon or rheumatologist. Before the injection you will have an injection with local anesthetic, which may sting a little, to numb the area around the knee. If needed the physician will drain excess joint fluid.

The injection of Arthrosamid® is a minimally invasive procedure that will be given to you as an outpatient.

As with any injection there is a small risk of infection when injecting Arthrosamid®. The physician will give you antibiotics 1-2 hours before the treatment to reduce the risk of infection. Consult your injecting physician about the antibiotic treatment.

1. Disinfection of the knee.

2. Injection using ultrasound.

3. Injection of Arthrosamid®.

4. Removal of the needle and placement of a patch at the injection site.

How does Arthrosamid® work?

Arthrosamid® integrates into the synovial tissue of the inner joint capsule of the knee and thickens the synovial membrane. Arthrosamid® provides durable cushioning of the inner capsular tissue, thereby reducing pain, decreasing joint stiffness, and improving function of the knee affected by osteoarthritis.

When should Arthrosamid® not be used?

Arthrosamid® should not be injected:

- If an active skin disease or infection is present at or near the injection site. If the knee joint is infected or severely inflamed.
- If you have previously received treatment with an absorbable intra-articular injectable (such as hyaluronic acid) that is not yet expected to be absorbed, according to manufacturer's information for the specific product.
- If you previously have received treatment with a different non-absorbable injectable/implant in the same knee.
- If you have received a knee alloplasty (knee replacement) or has any foreign material in the knee
- If you have undergone knee arthroscopy within the last 6 months.
- If you have uncontrolled hemophilia or are in uncontrolled anti-coagulation treatment.

Do not subsequently have an injection with a different non-absorbable implant in the injected knee.

What are the benefits?

As Arthrosamid® works to cushion the joint, it can reduce pain, decrease stiffness, and help movement. The hydrogel does not degrade over time and provides long-acting relief, improving quality of life. The expected performance lifetime is 5 years.

What are the risks/ side effects?

No serious side effects related to the hydrogel were reported in the clinical trials performed with Arthrosamid®. The most reported side effects were associated with the injection. These side effects were mild to moderate injection site pain and swelling during the first weeks to months following the injection and were generally transient in nature. Formation or swelling of Baker's cysts may also occur.

A rare side effect may be infection at the injection site. Contact the injecting physician if you experience any sign of infection. The infection must be treated **immediately** with antibiotics. Do not use corticosteroids, as this may complicate treatment of the infection.

Precautions

Arthrosamid® should be used with caution in patients: with acute or chronic infections in other parts of the body, with autoimmune disorders, in immunosuppressive therapy, with a compromised immune system, suffering from uncontrolled diabetes, undergoing major dental work or an invasive medical procedure.

Safety and effectiveness have not been established for: patients under 18 years of age, pregnant or lactating women, patients with BMI ≥ 35 kg/m², single doses higher than 6 mL.

What can I expect after the treatment?

The Arthrosamid® procedure is minimally invasive and typically takes about 10-15 minutes to perform. You will be able to go home or to work immediately after the injection.

As with any invasive joint procedure it is recommended to avoid strenuous activity e.g., tennis, jogging or long walks during the first few weeks after the injection and begin rehabilitation activities, when possible.

If you need surgery or dental work after the Arthrosamid® treatment, inform the treating physician that you have a permanent implant, and you should discuss the possible need for prophylactic antibiotic therapy.

Arthrosamid® is delivered with a patient implant card. The injecting physician will hand this out to you. It contains information regarding the implant. We recommend you keep this implant card for future reference.

If you experience any side effects, please consult the injecting physician.

In the unlikely event that removal of Arthrosamid® is required, it can be removed surgically.

Any serious incident that occurs in relation to the device should be reported to Contura International A/S at complaints@contura.com as soon as possible.

For more information and updates please refer to:
www.arthrosamid.com/patient.

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